

# ProScreen™ 6 Cup Instructions



✓ **STEP 1**

**Collect Specimen in cup –  
collector retains the lid**



✓ **STEP 2**

**Verify temperature is  
within range (32-38 C)**



✓ **STEP 3**

**Place the lid on the cap and make  
sure it's tightly sealed**



✓ **STEP 4**

**Place cup on its side, so it's  
resting on the "feet" in the  
cap.**


**Read results when the lines appear.**

**Wait 5 minutes before reading Positives/Non-negative results.**

**Do not read results after 10 minutes**


# ProScreen Results Interpretation

## Reading the Result Windows

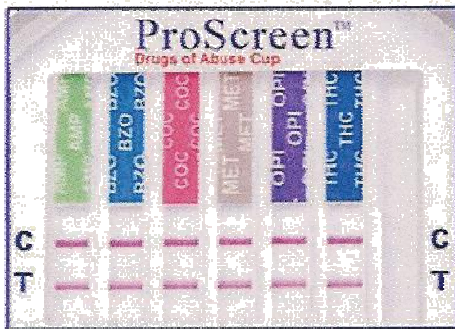


### NEGATIVE

A Negative test is indicated by the presence of a test line below each designated drug.

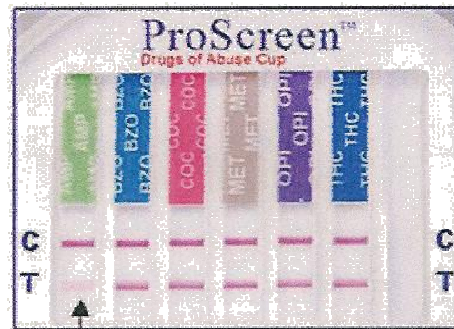


### NEGATIVE

This is also a Negative screen. Even a **very light** colored line indicates a negative result.

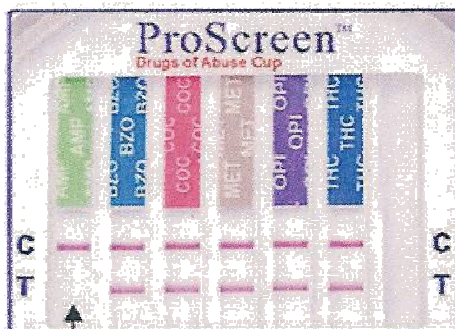

### POSITIVE/NON NEGATIVE

A Positive test is indicated by the **ABSENCE** of a Test Line below the designated drug, with the Control line present for that channel.

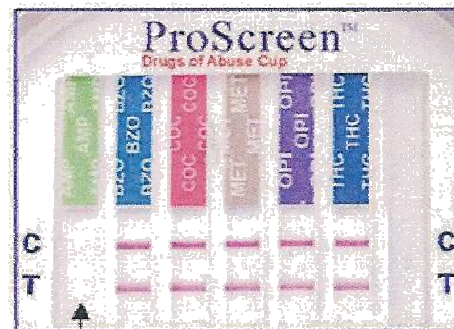

### INVALID

An Invalid test is indicated when the "C" Control Lines are completely missing in one or more of the test windows. In event of an invalid screen a second screen should be run.

*Wait 5 minutes before reading Positive Results. Do not read results past 10 Minutes.*

**Line Intensity (colour) is NOT important**